**EC REP** Shanghai International Holding Corp. GmbH  
Eiffestrasse 80, 20537 Hamburg, GERMANY

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## 9. Symboles standard

Attention, consulter les documents joints.

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## CONTRA-ANGLE GREEN 16:1

### 1. General discription

- Can be used for various dental clinical treatment and prevention.
- Can be connected to various E type handpiece that meet the ISO3964 standard.
- High-temperature sterilization or chemical disinfection.
- Should be professionally used by dentists as cutting tools for drilling or gripping teeth.

#### CAUTIONS

- Read this user's guide before use to fully understand the product functions and file for future reference.
- Prior to each use, all instruments and accessories must be inspected for proper operation.
- When using the handpiece, protect the patient at all time.
- Before each use, be sure that the bur is completely seated and locked in the instrument.
- Only use burs from reputable manufacturers with a shaft diameter of 2.348 mm~2.350 mm, and a max. total length of type 1: 22.6 mm; type 2: 44.6 mm; and type 3: 19.1 mm.
- After extended use, the handpiece may become noisy, replace the cartridge with a new one. Failure to replace the cartridge may cause accidents or impair operator's hearing.
- Handpieces should not be rebuilt. Replacing bearings could result in unexpected failure and patient injury due to different factors of new vs. old components.
- Do not extend burs outward from handpiece head. Burs must be inserted all the way into chuck for proper bur retention. Extended burs could result in bent burs, inadequate cooling or personal injury due to bur ejection.
- End of product life, the metal can be handled by separation and recycling disposal.
- Products storage and transportation should be carried out in environment with the following features: dry, stable (no collision), clean, normal temperature, normal pressure, away from acid gases and other harmful chemical substances, and humidity should be less than 90%.
- Maintenance should be made by professionals.

## 2. Technical Parameter

Description	E type low speed handpiece 16:1 Ratio, with internal cooling system
Atomization Air Pressure	245~392 Kpa (2.5~4.0 kgf/cm <sup>2</sup> )
Water Pressure	198 Kpa (2 kg)
Max Rotation Speed	2 500 rpm
Chuck Type	Three valvular reed
Grip Power	≥ 4 kg
Bur Applicable	Ø 2,334~2,355 mm
Noise	≤ 70 dB
Weight	Approx. 78,4 g

- Speed may vary slightly depending on the back-end configuration, the direction of rotation, and type of hose used.
  - This model can be used with standard internal water air motor and electric motor.
- The only burs to be used are those corresponding to the DIN EN ISO 1797-1 norm and are made from steel or hardened metal, with a shaft diameter of 2.334-2.350 mm, a total length of maximum 20 mm of contra-angle handpiece burs.

## 3. Assemble and disassemble

### 3.1. Assemble

Insert the contra-angle in the motor and rotate the contra angle or the motor until there is “click” sound where the spring insert goes inside to the groove on the motor.

### 3.2. Disassemble

Simply pull out the contra-angle from the motor.

**⚠ ATTENTION**

Do not fit or remove the contra-angle during rotation.

## 4. Care instruction

**⚠ ATTENTION**

Lubricate with oil spray after each use.

**Contra-angle head part**

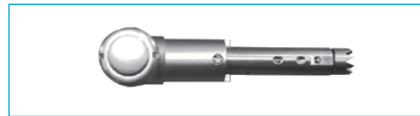
- Rotate the tension ring on the contra-angle until the end and take out the head of the contra-angle handpiece.

- Insert the turbine lubricant in the hole as illustrated. Lubricate one time each day, using the turbine lubricant 1~2 drops, in order to maintain a volume of 300/500 ml of lubricant.

- And also the heads should be lubricated after each use, take it out from the handpiece and lubricate the driving shaft with 1-2 drops of maintain oil.

## ATTENTION

Improper maintenance and care can lead to premature wear and malfunction.

## 5. Lubrication

Disconnect the contra-angle from the handpiece hose and supply a few drops of maintain oil for a few seconds into the drive air tube at the back end of the motor.

Lubricate after each use and/or before autoclaving. Spray one or two drops each time with turbine oil, or, spray for one or two seconds with maintenance oil.

## 6. Sterilization

The contra-angle is suitable for steam autoclaves. Sterilization is required first time you use it and after each patient as noted below.

### **Autoclaving Procedure:**

- ① Wipe off dirt with alcohol-soaked cloth.
- ② Lubricate with lubricant spray. Operate the handpiece for a few seconds after lubrication with turbine oil.
- ③ Place into autoclaving pouch and seal it in accordance with instructions on the pouch.
- ④ Autoclave the contra-angle at 134°C for minimum 18 min.

## 7. Warranty

The manufacturer grants the user a 12 months guarantee for its complete product range, except ball bearings (3 months guarantee) from the date of invoice issued. Maintenance over the term of guarantee will be at the customer 's charge.

The manufacturer will not be responsible for damage or injury resulting from:

- Excessive use.
- Improper manipulation of the product, or modification to product carried out by persons not authorized by the manufacturer.
- Failure to follow the instructions for installation, operation and maintenance of the handpiece.
- Damage of chemical, electrical or electrolysis due to improper autoclaving and storing.
- Improper working pressure.

The technical specifications and dimension in these instruction are given only as a guide. They may not be subject of any claim.

## 8. Information

The technical specifications, illustrations and dimension contained in these instructions are given only as a guide. They may not be the subject of any claim.

The manufacturer reserves the right to make technical improvement to its equipment without amending these instructions.

## 9. Standard symbols

Attention, consult accompanying documents

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