# LAB MOTOR

S-55

fr : Guide d'utilisation.....	02 - 07
en : User's guide.....	08 - 13
it : Manuale d'uso.....	14 - 19
es : Manual de usuario.....	20 - 25
pt : Guia do utilizador.....	26 - 31
de : Benutzerhandbuch.....	32 - 37
nl : Bedienungsanleitung.....	38 - 43

# LAB MOTOR S-55

fr

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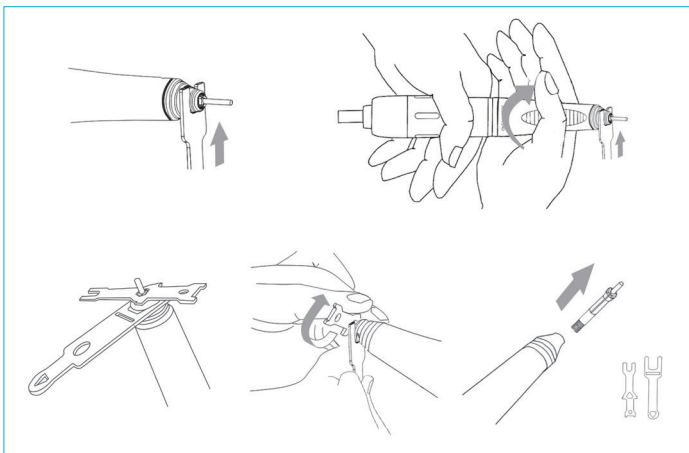
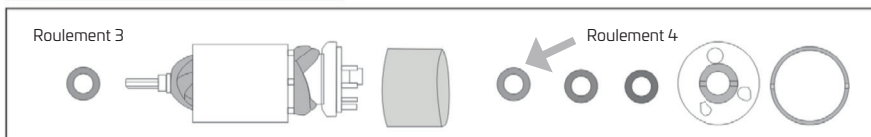
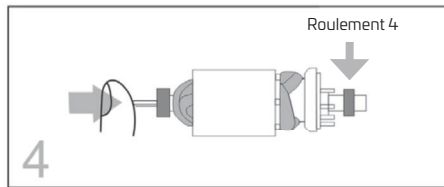
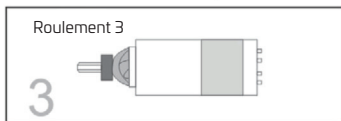
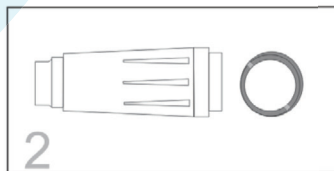
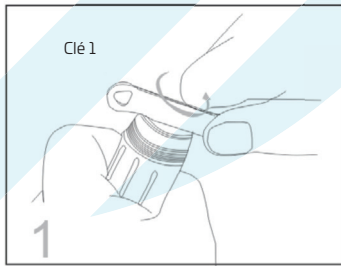
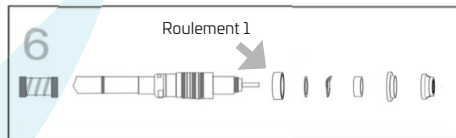
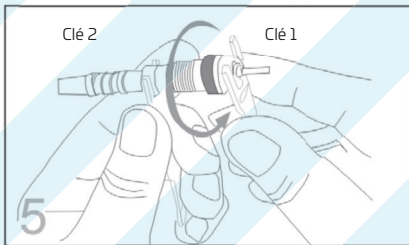
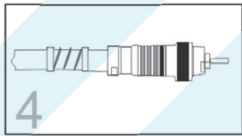
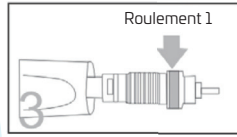
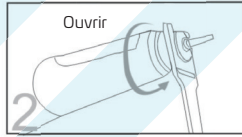
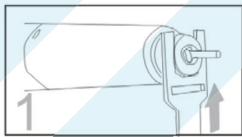
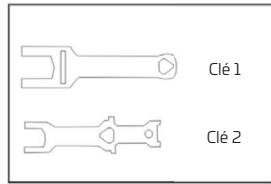
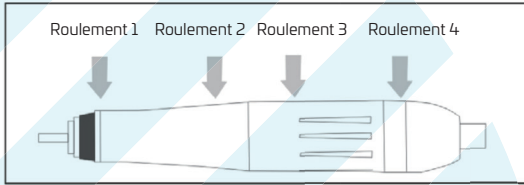
## BRUSHLESS MICRO MOTOR

en

### \*Read all instructions before using

- This motor is mainly designed for dental laboratory use.
- The manufacturer is not responsible for the accident causing by not following the instructions.
- Design and specification of the product can be subject to be changed for improving the quality.

**⚠ Note : Please wear protective equipment before operationb(protective goggles and safety gloves).**

### WARRANTY

- Handpiece, control box and foot pedal: 1 year.
- Warranty does not include misuse or normal wear of bearing, collect chuck.

### SERVICE CHARGE

In case of the fault or malfunction from the misuse of the user:

1. Using with wrong input Voltage
2. In case of user's misuse or improper use.
3. Dropping down during setting or moving.
4. Using consumable parts or accessories not recommended by manufacturer.
5. Malfunction after repairing at the places not recommended by manufacturer.
6. Damage caused by improper use.

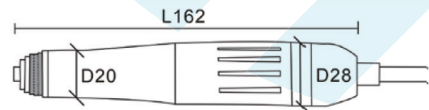
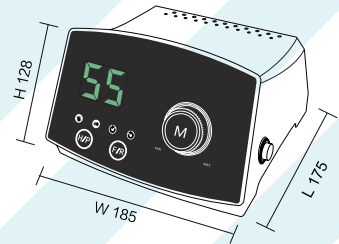
### PRODUCT SPECIFICATION

#### • CONTROL BOX:

INPUT POWER SOURCE	FUSE	WEIGHT	MAXIMUM LOAD CURRENT	DIMENSION(mm)
Ac100 ~120V 50/60Hz	5 A	2.53 kg	9 A	W 185 L 175 H 128
Ac200 ~240V 50/60Hz	3 A			

#### • HANDPIECE:








OUTPUT	240 w
MAXIMUM SPEED	55,000 rpm
TORQUE	8.5 N.cm
WEIGHT	245 g
DIMENSION	L162 Ø20-28 mm

### PART NAME:

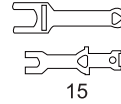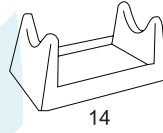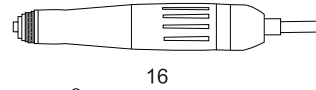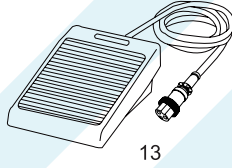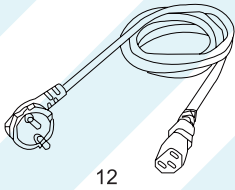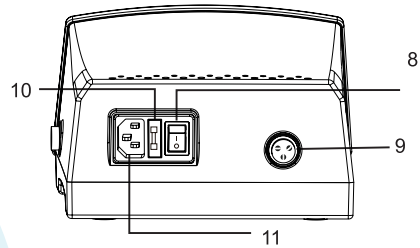
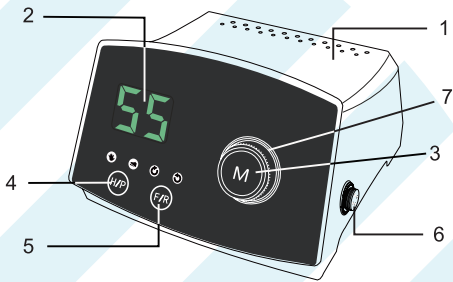
1. Control box
2. RPM display
3. Speed control dial motor "on" or "off" button
4. Foot pedal "on" or "off"
5. Forward or reverse button
6. Motor connector
7. Left right turning direction (green halo is forward / red halo is reverse)
8. Power switch
9. Foot pedal connector
10. Fuse holder
11. Power connection
12. Power cord
13. Foot speed control pedal
14. Handpiece holder
15. Wrench
16. Handpiece

## SETTING & OPERATION

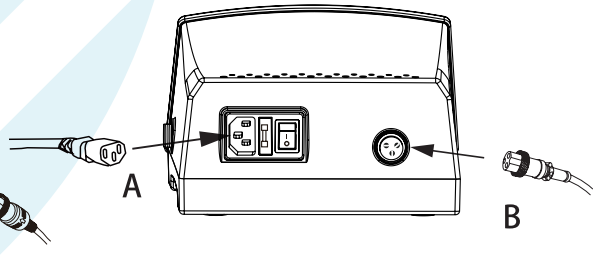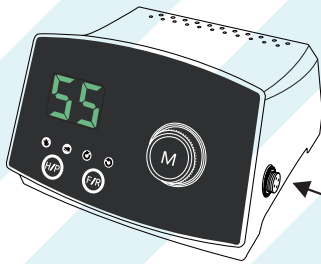**⚠ WARNING:** Please make sure that power is off and connect the motor and root pedal. (Otherwise, it might cause malfunction).

- After checking off switch in control box, plug in:
  - Connect the power cord with control box. Picture A.
  - Connect the foot pedal to control box. Picture B.
  - Connect the handpiece to output of control box. Picture C.

2. As indicated by the arrows, it lights green on the power on and shows the figures on display screen. 

- As indicated by the arrows, the green light is gone off when power off.  
- The figures are disappeared.

3. Left Right turning direction (green halo is positive red halo is inversion).  
The  light is on when turning right, the  light is on when turning left.

- Manual control operating instructions.
  - Adjust the speed rotating-button to "0".
  - Turn on the power while holding the handpiece.
  - Rotating handpiece switches to see if shank is locked (see picture F)

**Note :** if the collect chuck is in the open states, machine does not work.

- Select manual control mode.
- Press start "m" to start the motor,  left & right turning direction halo lights up   
Lowly adjust speed rotating-button to see if handpiece work. No abnormal condition, adjust dill bit, to required speed.


## Foot control operating instruction

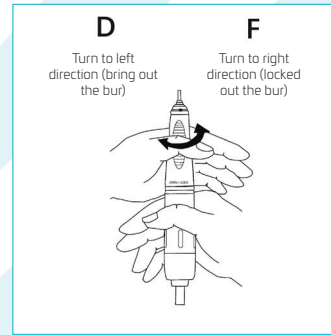- Adjust the speed rotating-button to maximum speed.
- Turn on the power while holding the handpiece.
- Rotating handpiece switches to see if hank is locked (see picture F).

**Note: if the collect chuck is in the open state, machine does not work.**

- Select foot control mode 
- Left & right turning direction halo light, up. 
- Foot pedal, lowly adjust, the speed check if handpiece work properly.

No abnormal condition, adjust drill bit, to required speed.

5. The handpiece is protected and stopped working when it is overworked. The screen shows (E4) or (E6) an overheat protection error. switch of to remove overload protection, turn on the machine again to continue working.

## SUGGESTION FOR MAINTENANCE

1. Keep the place around in handpiece clean from dust and grinding.
2. Remove dust on handpiece wiping with isopropyl alcohol and avoid wet inside.
3. Do not apply the oil to the good or clean water. The grease inside of the bearing could melt away due to the oil.
4. Air cleaning is acceptable, but air pressure may affect to the handpiece badly.

## FAILURE AND SOLUTION

If the product does not work after checking, please contact the distributor or manufacturer.

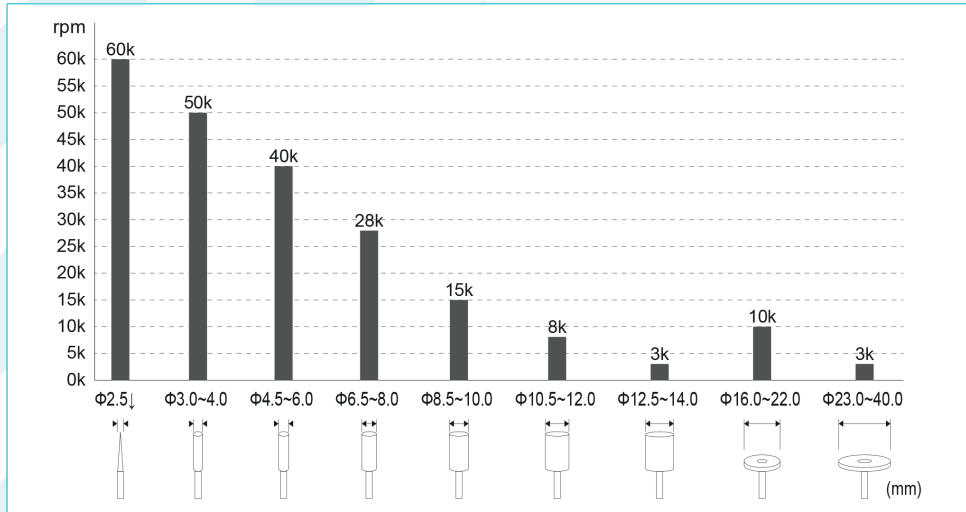
CONDITION	CHECK POINT	SOLUTION
RPM display doesn't appear	Power, fuse, power switch	Check to replace the accessory
Error code	E1/E5 Hardware protection	Replace the mainboard
	E2/E4 Overload protection	Shutdown & restart
	E3 Voltage protection	Check if voltage meet machine requirement
	E6 Temperature protection	Check if handpiece overworks for long period
Buttons are not working	Check if it doesn't work after beep sound.	Ask to the distributor or manufacturer
Handpiece heats up	The handpiece is warm	It heats up when it is used continuously, take a break until the heat disappear
	The handpiece is hot	When it's used with overload continuously, it heats, up.
Handpiece doesn't rotate	Check if the chuck is open. (occur E4)	Lock the chuck
	Check if handpiece is well connected in the connector. (occur E4)	Connect well again the handpiece to the connector and if still shows E4 continuously, contact the distributor or manufacturer

## ⚠ CAUTIONS

1. Please keep the RPM according to below diagram of right against the accident.

The deepest depth recommended for grinding head to insert into collet chuck is 27 to 30 (mm). Longer the grinding head exposed centrifugal force, the weaker of the grinding head and easily cause the rod body bended. Serious accident, might happen under excessive speed while using large diameter grinding head.

Such as: bending of the grinding head bar (which may hurt fingers), bearing damage, collet chuck deformed and so on.


2. Protective glasses and gloves must be worn when operating the appliance.
3. If any problem occurs in handpiece, use after repairing.
4. Make sure that the switch is off before putting the power plug in the socket, in order to avoid an unexpected injury due to sudden start.
5. A circuit breaker is provided to protect the motor. It is actuated when there is an overload which the motor cannot stand. Work within a range of load which it is not actuated.
6. Do not open the chuck while operating.
7. Keep the handpiece with setting a bur to the collet chuck after operation.
8. Do not put the handpiece down before it stops completely.
9. Be sure to pull out the power cord before moving the control box.
10. Use the specified voltage.
11. Use accessories suitable for this product.
12. Wear proper protective clothes and equipment, including a dust mask and safety goggles. Use a proper suction unit and a ventilator.
13. Keep the workplace clean. Debris may cause an accident.
14. Memory saves the condition of direction and hand foot election on turning of the power, therefore, be careful when the power on.
15. Do not work at dangerous place. Do not use the control box in humid place.
16. Check for any trouble, including vibration of bur at a low speed and then start high speed.
17. When RPM is zero, you can replace the bur.
18. Do not turn on the power of control box while the collet chuck of handpiece open. otherwise, the figure display will show (E4).


## SAFETY CAUTIONS

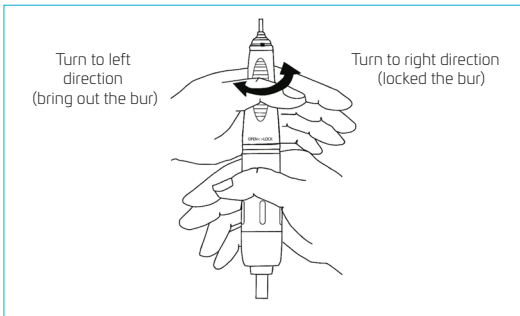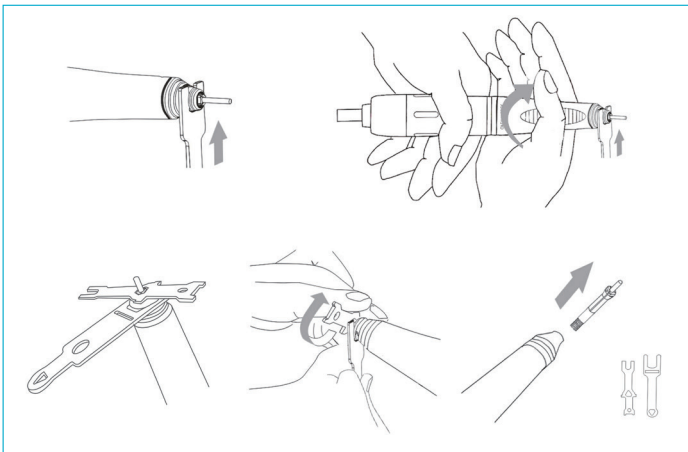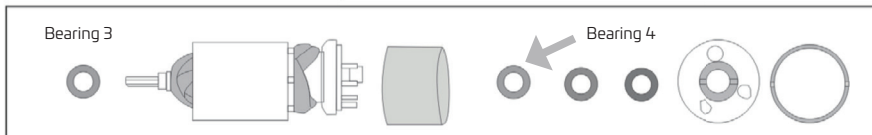
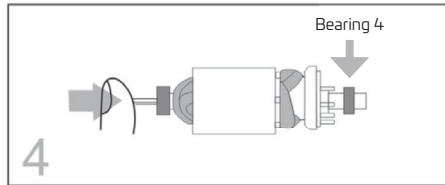
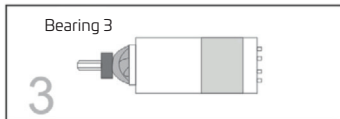
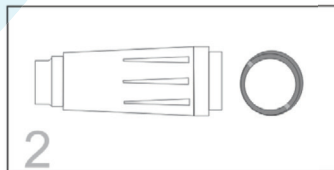
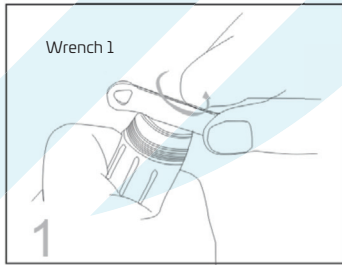
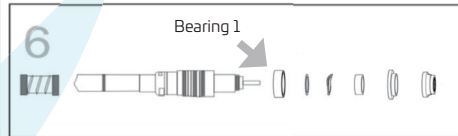
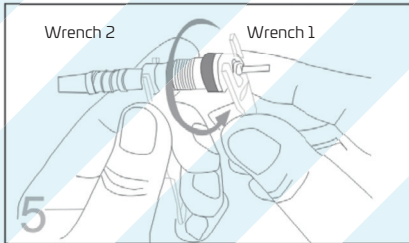
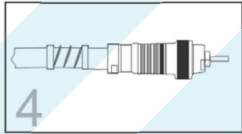
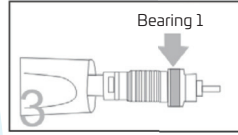
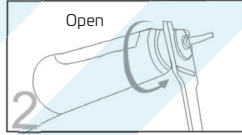
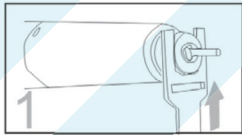
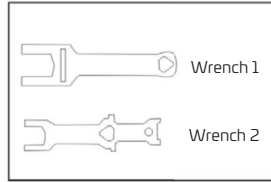
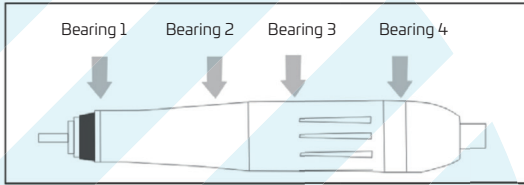
1. Do not use with damaged or faulty power cord, plug and outlet. Maybe it can cause the danger of fire, electric shock.
2. Do not use the broken power plug. Otherwise, it causes the danger of electric shock and fire.
3. When unplug the power cord, hold the plug and do not touch with wet hand.
4. Do not bend the power cord excessively and do not press with heavy things. It may cause electric shock and fire.
5. Do not place near heat equipment and candles or cigarette causing fire.
6. Put this product into someplace where children cannot be touch with.
7. Use the grounding conductor. Breakage or electric leak may cause the danger of electric shock.
8. Please do not use many high-power electronic devices in one socket, otherwise, it may cause electric shock or fire.
9. Do not place or set up the product in the humid and rainy location. It brings the danger of electric shock and fire.
10. Do not drop or damage the handpiece. In case of malfunction, dropping, drowning, do not use and contact to the manufacturer.
11. Use the products in proper use as per the instruction.
12. You cannot change the functions of products casually.
13. The manufacturer is not responsible for the accident causing by not following the instructions.
14. Please wear protective equipment before operation (protective goggles and safety gloves).

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Development Co., Ltd.**